

INSTRUCTION MANUAL

CA-A3211E

Oxygen cylinder holder

7C02126100A3

Preface


Thank you for purchasing our **oxygen cylinder holder**.

This instruction manual describes precautions and how to safely use and operate the **oxygen cylinder holder**.

- **To ensure safe and proper use of the oxygen cylinder holder, be sure to read this instruction manual and the instruction manual for the suitable bed before use.**
- The safe and correct operating procedures should be explained not only to the person using the bed with **oxygen cylinder holder** for medical treatment but also to the user's caregiver.
- After reading this instruction manual, store it in a place that can be easily accessed for later reference. Users and/or patients are requested to report information about any serious incident that occurs within the EU to the regulating authority of the member country of residence. Serious incident means any incident that directly or indirectly resulted in, may have resulted in, or may result in any of the following.
 - Death of a patient, user, or other person
 - Serious temporary or permanent adverse effects on the health of a patient, user, or other person
 - Serious public health threat
- Some of the descriptions in this instruction manual may differ from the product you have purchased due to model improvements.
- Values described for the operation range, dimensions, angles, and weights are subject to normal manufacturing tolerances. Except for special cases, words such as approximately and about are omitted in this instruction manual to improve readability.
- For any queries, contact Paramount Bed (see p. 4) or your distributor.


Intended use

- This oxygen cylinder holder is designed to be attached to the bed as being able to use an oxygen cylinder.
- Intended users: Medical professionals
- Patient target group: Refer to the instruction manual of the medical device to be used together with this product.

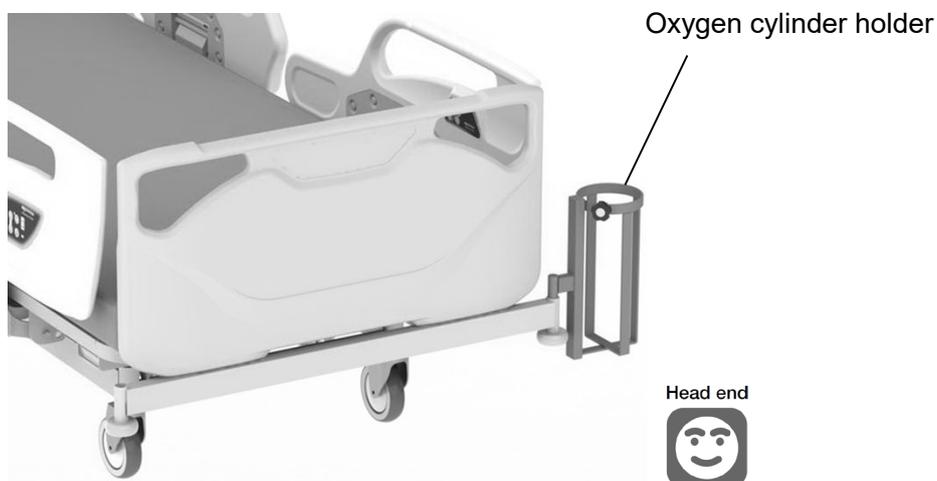

Part names and part confirmation

- After unpacking, check that there are no missing or damaged parts. If any parts are missing or damaged, contact Paramount Bed or your distributor.

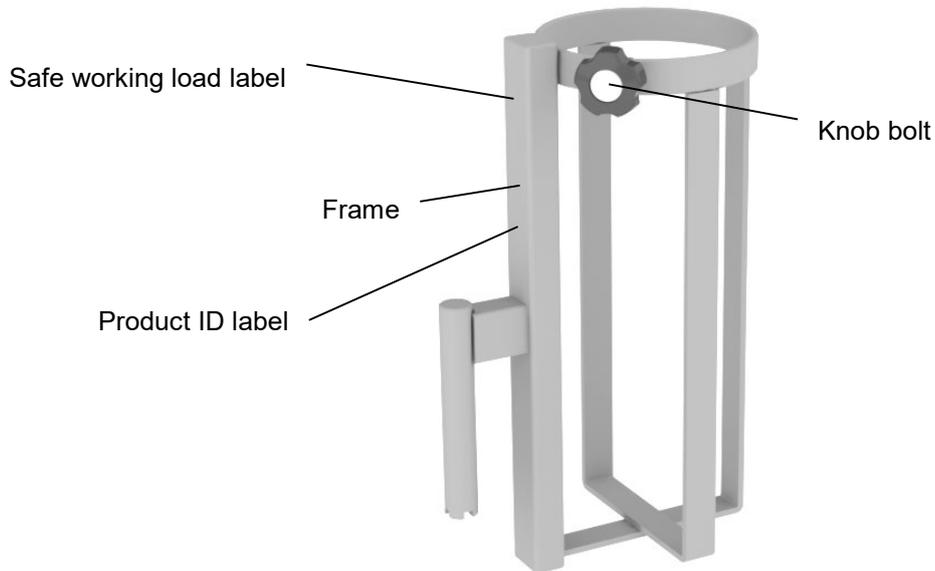

Specifications

<Specifications>

Subject to normal manufacturing tolerance

Product name	Oxygen cylinder holder
Model No.	CA-A3211E
External dimensions (mm)	195 x 133 x 325
Product weight (kg)	1.6
Materials	Made of steel, chrome plating (part is resin mold item)
Outside diameter of usable oxygen cylinder	500ℓ (95mm - 115mm dia.)
Load capacity (kg)	15

Environmental conditions	Range	
	When transported or stored	When in use
Temperature	-10°C to 50°C	5°C to 40°C
Humidity	20%RH to 90%RH at 30°C	20%RH to 90%RH at 30°C
Air pressure	700hPa to 1060hPa	700hPa to 1060hPa

<Classification>

Classification as medical device in Europe: Class 1

<Applicable standards>

Since this device is an accessory, for safety standards applied on this device, refer to the instruction manual of the suitable medical devices.

<Product lifetime>

The lifetime of this equipment is typically eight (8) years.

Lifetime is defined as the period during which the product will maintain the specified performance and safety, provided it has been maintained and operated in conditions of normal use in accordance with the requirements in this instruction manual.


Attaching the oxygen cylinder holder

WARNING

- Make sure that the knob bolt is firmly tighten. There is a risk of insufficient oxygen supply.
- When using the oxygen cylinder, take care to prevent it from coming into contact with any objects in the vicinity of the bed. There is a risk of an injury by damage of a flow meter or an oxygen cylinder if the cylinder comes into contact with another object.
- Load capacity is 15kg. Do not insert the oxygen cylinder exceeding the load capacity because it may damage the oxygen cylinder holder, resulting in injury.

■ The oxygen cylinder holder can be used by inserted into one of the option attachment holes of the bed.

- ① Insert the oxygen cylinder holder into one of the option attachment holes of the bed.
- ② The oxygen cylinder holder can be set in two positions as shown in figure. After setting, make sure that the oxygen cylinder holder does not rotate.
- ③ Loosen the knob bolt on the oxygen cylinder holder, and then slowly insert the oxygen cylinder into the holder from the top.
- ④ Firmly tighten the knob bolt to fasten the oxygen cylinder.

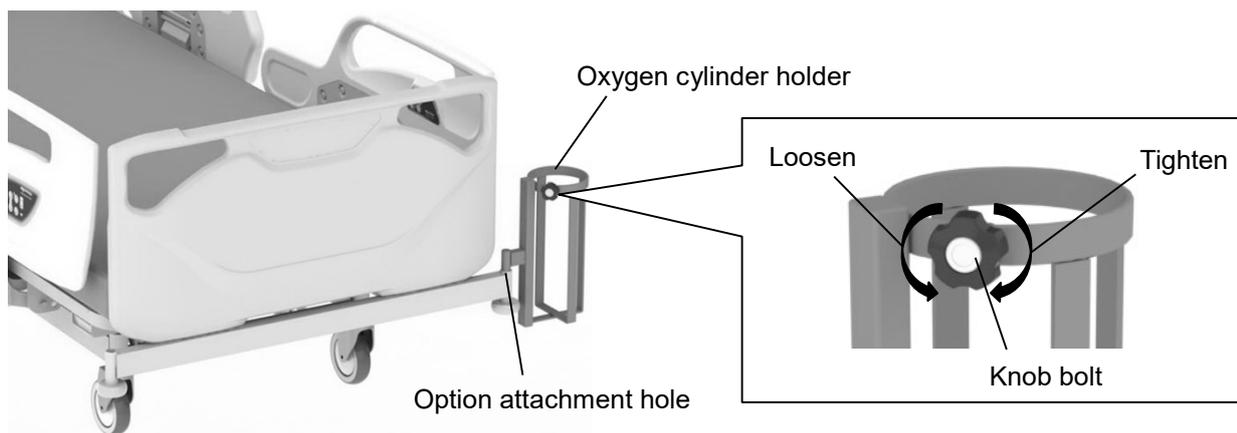


Daily maintenance/Long-term storage/Disposal

<Daily maintenance>

- To disinfect parts.

Use one of the disinfectants listed below at the specific concentration, making sure to follow the instructions in the manuals or other materials available for the selected disinfectant.

76.9% - 81.4%	ethanol for disinfection
0.05% - 0.2%	benzalkonium chloride (e.g. Osvan)
0.05% - 0.2%	benzethonium chloride (e.g. Hiamine)
0.05%	chlorhexidine gluconate (e.g. Hibitane)
0.02% - 0.05%	sodium hypochlorite (e.g. Milton)

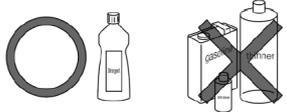 CAUTION	
<ul style="list-style-type: none"> ● Do not use volatile cleaning fluids (such as paint thinner, benzene or gasoline) and cresol. Discoloration or deterioration could result. 	



- The product cannot be used with an ozone sterilizer or an autoclave sterilizer, etc.

<Long-term storage>

- Do not place location that tend to be exposed to high temperatures, high humidity, low temperatures, dry or dusty conditions. Store at a temperature between -10°C and 50°C, humidity between 20%RH and 90%RH at 30°C, and atmospheric pressure between 700hPa and 1060hPa.
- Keep the instruction manual insert in a safe place so that they do not become lost.

< Disposal >

Dispose of the product in accordance with local laws and regulations. Contact Paramount Bed or your distributor (see "Contact information" as shown below) for any queries.

Product symbol definitions

- The symbols below are used for this product.

Symbol	Description	Symbol	Description
	Date of manufacture		Serial number
	Manufacturer for CE mark		Country of manufacture (Made in China)
	CE mark		Model number
	Authorized representative in the European community.		Refer to the instruction manual
	* Either of the symbols is used.		Medical device (This product is an accessory for medical devices and such accessory is required to be labeled as "Medical Device" under medical device regulations in Europe.)
	Safe working load		

After-sales service

1. When requesting repairs

Contact Paramount Bed or your distributor (see "Contact information" as shown below) for repair.

■ Details necessary when requesting repair

- Product name, model number, safety label number
- Date of purchase
- Details of fault or problem (please provide as much detail as possible)
- Your name, organization, address, and phone number

2. Minimum stock availability period

Paramount Bed keeps replacement parts (necessary to maintain oxygen cylinder holder performance) in stock for at least 8 years after the discontinuation of production.

3. Questions regarding after-sales service

Contact Paramount Bed or your distributor (see "Contact information" as shown below).

Contact information

For any queries, contact to the address or telephone number listed below.

[Location in Japan]	[Location in China]
 <p>PARAMOUNT BED CO., LTD. 14-5, 2-chome, Higashisuna, Koto-ku, Tokyo, 136-8670, Japan TEL: +81-(0)3-3648-2961 FAX: +81-(0)3-3648-5781</p>	<p>PARAMOUNT BED (CHINA) CO., LTD. Plot A-105, Wuxi National Hi&New Tech Industrial Development Zone, Wuxi, Jiangsu, 214028, China TEL: +86-510-85323205 FAX: +86-510-8532-3212</p>



NOTES

NOTES

